


Control Layout

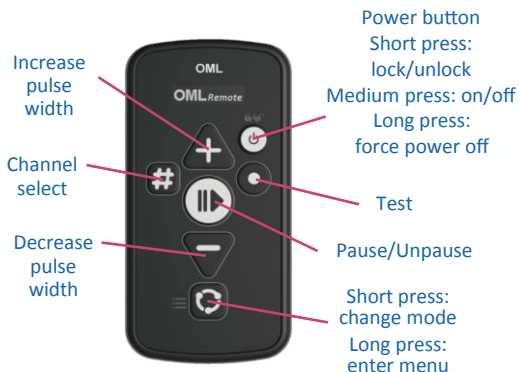

Join a Remote and a Pace XL (Part 1)

- 1 Turn on the Remote by pressing
- 2 Enter the options menu by long pressing the button
- 3 Press again until the display shows the wireless symbol that is highlighted
- 4 Select the wireless option by pressing
- 5 The display now shows that it is seeking Pace XL device to connect to.

Join a Remote and a Pace XL (Part 2)

- 6 Turn on your Pace XL device by pressing the control dial
- 7 Press the control dial down until the user accessible menu is shown on the display
- 8 Turn the dial the until wireless options is shown then select it
- 9 Turn the dial until the join option is shown then select it
- 10 The Pace XL is now seeking the nearest OML Remote that is also in joining mode.

SUCCESS!

Your OML Remote and Pace XL are joined. If wanting to join a second Pace XL follow the steps in the Joining a second Pace XL the Remote section. If using a single Pace XL the Remote section. If using a single Pace XL press the pause button to exit the menu.



Join a second Pace XL and Remote

- 1 If intending to join a second Pace XL, do not press the pause button to exit the joining process after the first Pace XL.



- 2 The OML Remote will automatically search for a second Pace XL



- 3 Repeat the Part 2 steps to join the second Pace XL



- 4 The OML Remote will connect and assign it as device 2. Press pause to exit the menu

SUCCESS!



Setup advice

- The OML Remote requires a Pace XL with software from at least V1.4.02 to work.
- During any connection attempt ensure the devices are close together for a strong signal.
- During setup the OML Remote will look for a Pace XL device for up to 2 minutes whilst the Pace XL will search for 1 minute.
- When a Remote is joined to a Pace XL, any previously joined Linqs are wiped from memory and must be established.
- Make sure the Linq does not have a battery before starting this joining process.
- During airplane mode communication between the Remote, stimulator, and Linq is disabled.

Join the Pace XL and a Linq

- 1 If using a Remote make sure it is unlocked by pressing the power button. Make sure there is also no battery in the Linq.



- 2 Press the control dial down on the Pace to enter user accessible menu



- 3 Use the dial to find the join F.switch option and select it



- 4 The Pace XL is now seeking the nearest Linq also in joining mode.



- 5 Place a battery into the Linq, close the cover, then press the joining button



SUCCESS!

Your Linq is now joined to your Pace XL confirmed by a notification on the Pace XL display.

