


Setting up the stimulator for dropped foot

To enter setup mode:
(Pace must be in Pause mode)

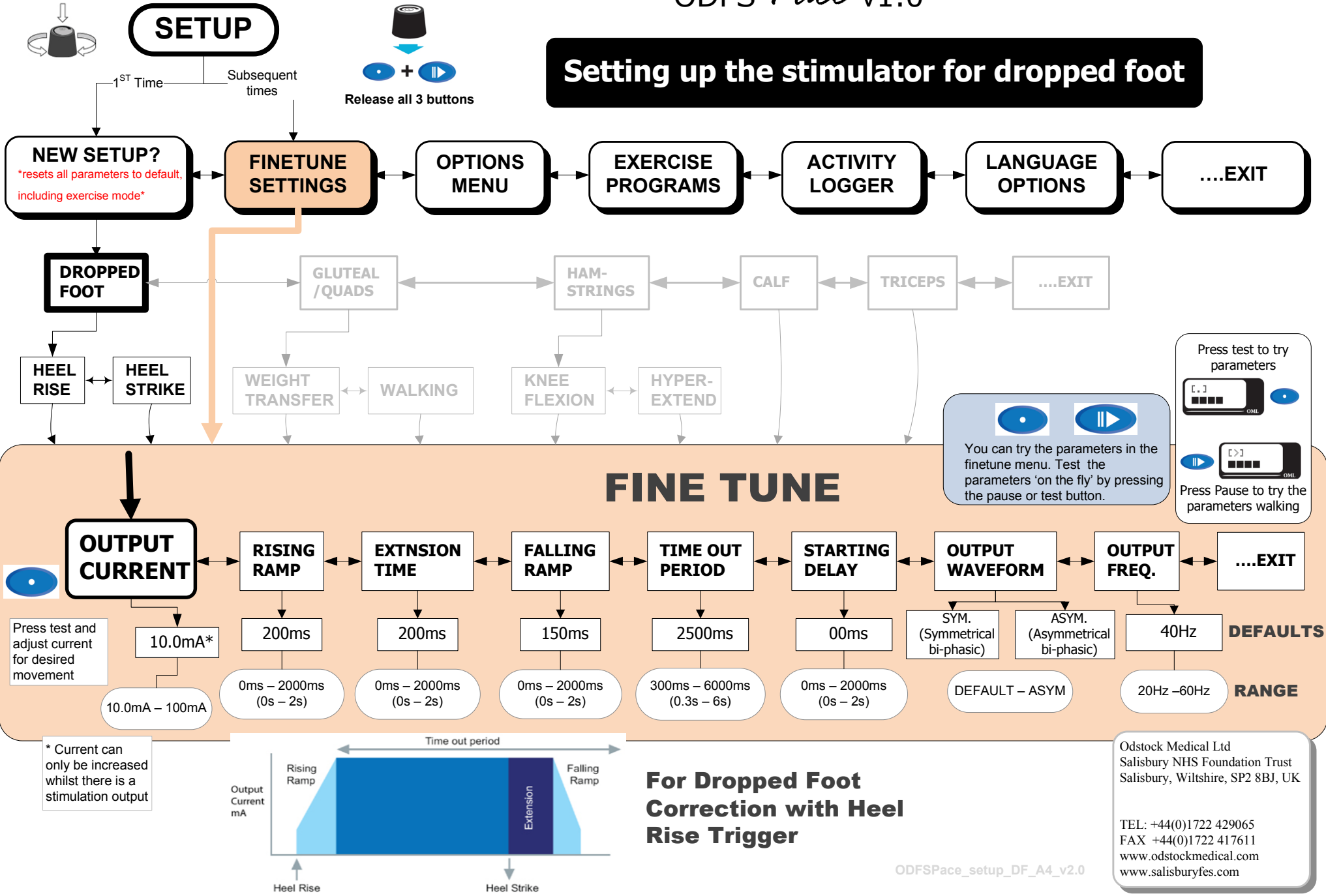


You can try the parameters in the finetune menu. Test the parameters 'on the fly' by pressing the pause or test button.

Press test to try parameters
Press Pause to try the parameters walking

Press test and adjust current for desired movement

* Current can only be increased whilst there is a stimulation output

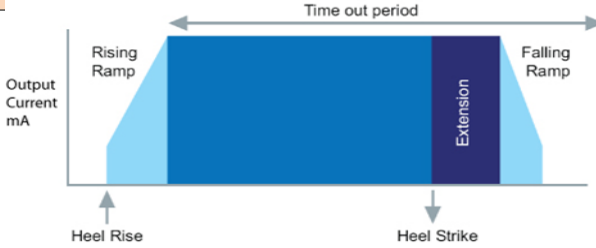

For Dropped Foot Correction with Heel Rise Trigger

Odstock Medical Ltd
Salisbury NHS Foundation Trust
Salisbury, Wiltshire, SP2 8BJ, UK

TEL: +44(0)1722 429065
FAX +44(0)1722 417611
www.odstockmedical.com
www.salisburyfes.com